# Dowing<sup>2</sup>

## Quick Start Guide / Snelstartgids

ID manual: 6o628o  
Revision / Revisie: 1.o.o  
Date / Datum: 01-05-2020

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This is the Quick start guide for Dowing<sup>2</sup>, a dynamic arm support system. This product is developed, manufactured and potentially distributed by Focal Meditech B.V. This quick start guide contains the basic information regarding Dowing<sup>2</sup>. The aim of this information is to ensure successful, safe and effective use of the device. This quick start guide will give you a quick look on the features and important safety warnings of Dowing<sup>2</sup>. Please take note of the safety warnings before using Dowing<sup>2</sup>.

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## Overview / Overzicht

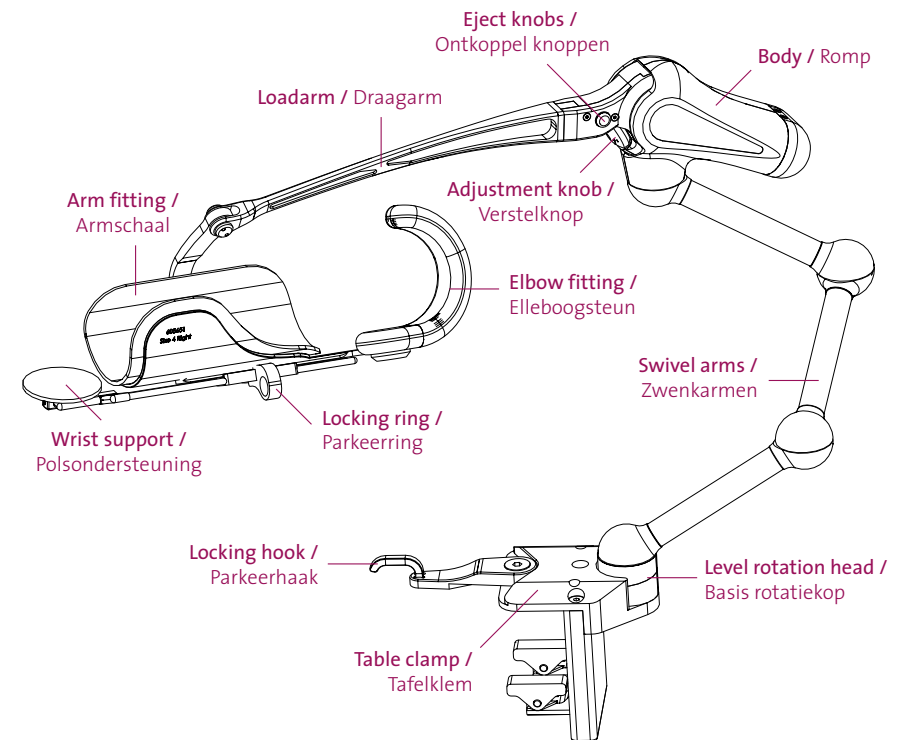
Figure 1: Overview Dowing<sup>2</sup> with terms of specific parts  
Figuur 1: Overzicht Dowing<sup>2</sup> met benaming specifieke onderdelen

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If Dowing<sup>2</sup> is not used for a short time it is recommended to place the Dowing<sup>2</sup> in its intended parking bracket. This will provide a safe situation for both Dowing<sup>2</sup> and its surrounding. The same procedure is followed for the table clamp and the chair / stand clamp.

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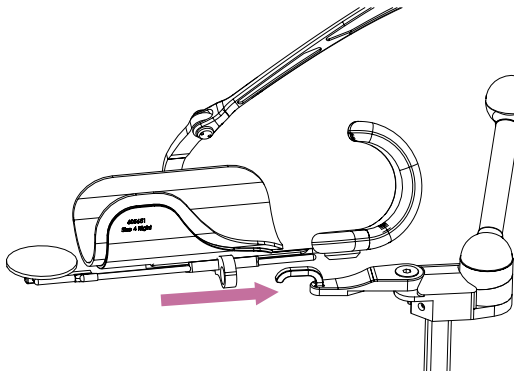
Figure 2: Park Dowing<sup>2</sup>  
Figuur 2: Parkeer Dowing<sup>2</sup>

### Park Dowing<sup>2</sup>

**STEP 1:** Move human interface with the locking ring backwards over the locking hook.

**STEP 2:** Move the arm with human interface up until the balance becomes less.

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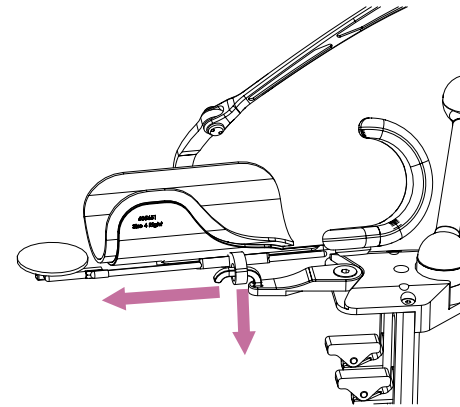
Figure 3: Start using Dowing<sup>2</sup>  
Figure 3: Start using Dowing<sup>2</sup>

### Start using Dowing<sup>2</sup>

**STEP 1:** Position the user's arm in the arm fitting.


**STEP 2:** Apply a small force downwards and slide the locking ring forwards over the hook.

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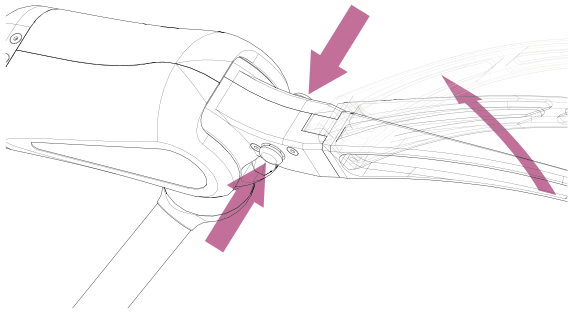The human interface including the loadarm, arm fitting, elbow fitting and wrist support can be removed easily. To disconnect the Human Interface, press both red buttons at the top of Dowing<sup>2</sup>, and move the Human Interface away from the body in the upwards direction.

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
Replacing the human interface can be done by pushing the Human Interface into the body of Dowing<sup>2</sup>. The buttons do not have to be pushed. When the lever is in position, it cannot be removed without pressing the buttons.

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If Dowing<sup>2</sup> is not used for a long time it is recommended to detach the Dowing<sup>2</sup> from its bracket. This will provide a safe situation for both Dowing<sup>2</sup> and its surrounding.

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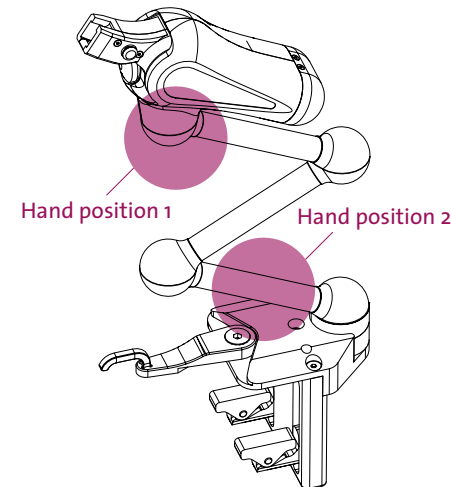### Detach Dowing<sup>2</sup>

**STEP 1:** Lock Dowing<sup>2</sup> in the parking position (see park or start using Dowing<sup>2</sup>)

**STEP 2:** Remove the user's arm

**STEP 3:** Remove Dowing<sup>2</sup> from the parking position. Warning: the arm fitting will move upwards due to the energy stored in the spring.

**STEP 4:** Remove the human interface (See attach and detach the human interface)

**STEP 5:** Hold with one hand the body of Dowing<sup>2</sup>; with the other hand, lift up in a straight line the lower swivel arm from the clamp

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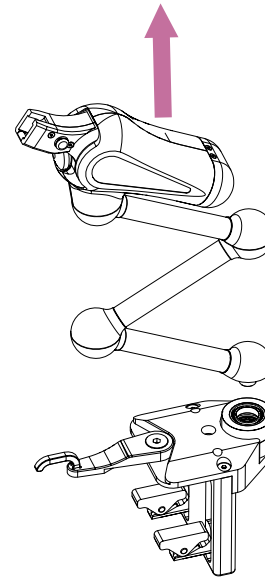
*Figure 6: Take Dowing<sup>2</sup> apart from the table clamp or chair mount*

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### Attach Dowing<sup>2</sup>

**STEP 1:** Position Dowing<sup>2</sup> above the clamp.

**STEP 2:** Insert the lower swivel arm into the rotation head.

**STEP 3:** Attach the human interface.

**STEP 4:** Lock Dowing<sup>2</sup> in the parking position.

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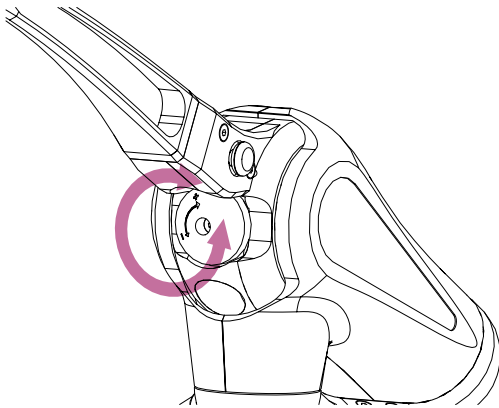
*Figure 7: Adjust the balancing force*  
*Figuur 7: Aanpassen balanceerkracht*

The upwards balancing force can be adjusted by turning the adjustment knob. The force required to rotate the adjustment knob can change depending on the position of the human interface. This is normal behaviour. Because of this, the system can be adjusted using little force. Rotating the knob clockwise will give more balancing force, counter clockwise will give less force.

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The complete user manual is available on our website and can be downloaded (<http://www.focalmeditech.nl/en/documentation>). In case you want to receive a printed user manual in your own language, please contact us at +31 (0)13 533 31 03 and we will send you without any costs a hard copy.

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