Dowing²

Quick Start Guide / Snelstartgids

ID manual: 606280
Revision / Revisie: 4.0
Date / Datum: 01-05-2024

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This is the Quick start guide for Dowing², a dynamic arm support system. This product is developed, manufactured and potentially distributed by Focal Meditech B.V. This quick start guide contains the basic information regarding Dowing². The aim of this information is to ensure successful, safe and effective use of the device. This quick start guide will give you a quick look on the features and important safety warnings of Dowing². Please take note of the safety warnings before using Dowing².

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Overview / Overzicht

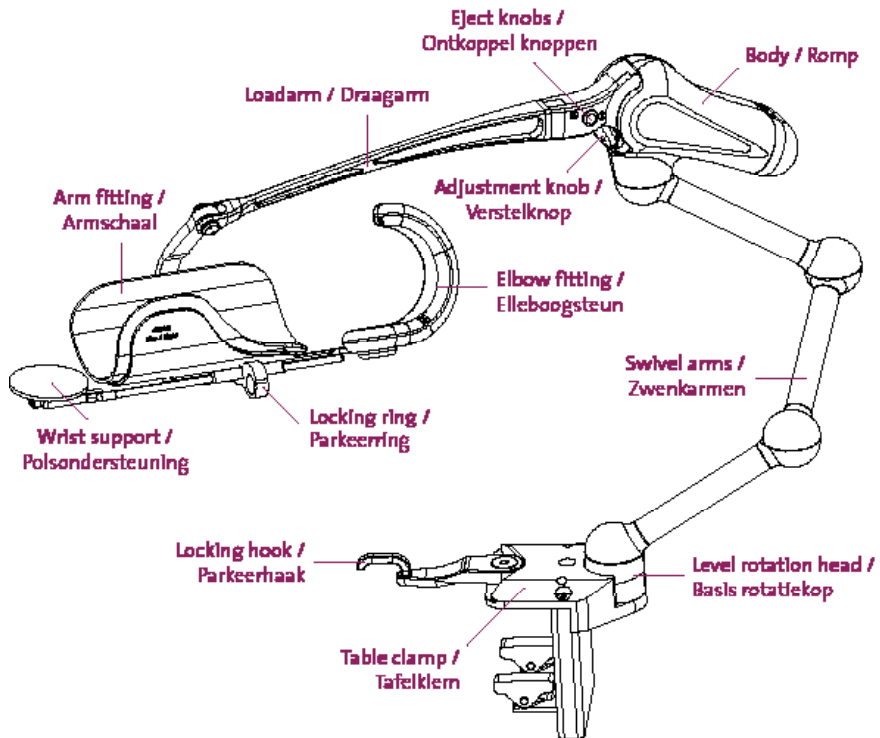
Figure 1: Overview Dowing² with terms of specific parts

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Safety notices / Veiligheidsnotities

Danger: Prevent direct contact with water or any other liquid.

Failure to prevent this can lead to malfunctioning of device or bodily harm.

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Danger: Prevent exposure to extreme temperature (see environment conditions). Failure to prevent this can lead to malfunctioning of the device or bodily harm.

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Danger: Mechanical energy is stored for balancing the arm. During non-intended removal of the arm from the arm fitting of Dowing², this mechanical energy will be released resulting in a fast moving Human Interface which can result in bodily harm.

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Warning: Do not modify any part of this equipment without authorization of Focal Meditech. This can lead to malfunctioning and will void the warranty.

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Warning: In case of faulty device contact Focal Meditech or your dealer. Do not try to fix it yourself. Failure of this will void the warranty.

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Warning: In case of doubt about safety of the device contact Focal Meditech or your dealer.

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Warning: In case of a serious incident when using the device, contact Focal Meditech and the national authority of your country.

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Warning: For safety and functional reasons, Dowing² may only be installed under the operating conditions described in the user manual. Dowing² may only be mounted on a (electric) wheelchair under strict conditions, which are described in more detail in the user manual.

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Park or start using Dowing² / Parkeer of start gebruik Dowing²

If Dowing² is not used for a short time it is recommended to place the Dowing² in its intended parking bracket. This will provide a safe situation for both Dowing² and its surrounding. The same procedure is followed for the table clamp and the chair / stand clamp.

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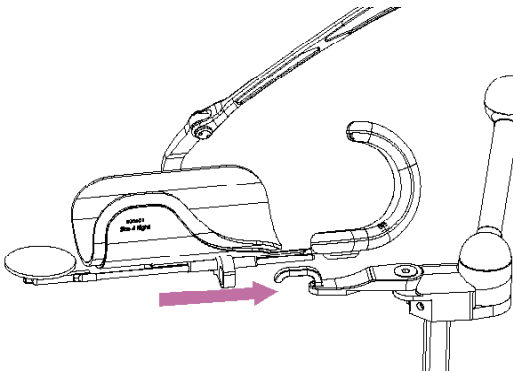
Figure 2: Park Dowing²
Figuur 2: Parkeer Dowing²

Park Dowing²

STEP 1: Move human interface with the locking ring backwards over the locking hook.

STEP 2: Move the arm with human interface up until the balance becomes less.

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Figure 3: Start using Dowing²

Start using Dowing²

STEP 1: Position the user's arm in the arm fitting.

STEP 2: Apply a small force downwards and slide the locking ring forwards over the hook.

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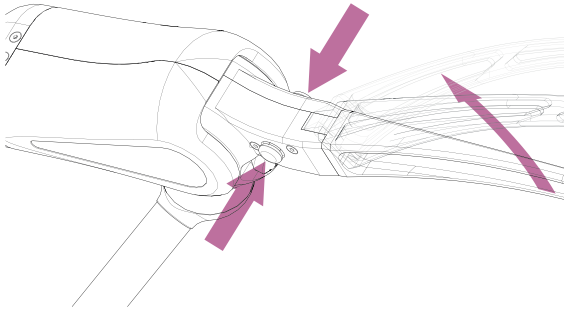
Figure 4: Attach / detach human interface

Figuur 4: Koppelen / ontkoppelen human interface

The human interface including the loadarm, arm fitting, elbow fitting and wrist support can be removed easily. To disconnect the Human Interface, press both red buttons at the top of Dowing², and move the Human Interface away from the body in the upwards direction.

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Replacing the human interface can be done by pushing the Human Interface into the body of Dowing². The buttons do not have to be pushed. When the lever is in position, it cannot be removed without pressing the buttons.

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Attach / detach Dowing² / Koppelen / ontkoppelen Dowing²

If Dowing² is not used for a long time it is recommended to detach the Dowing² from its bracket. This will provide a safe situation for both Dowing² and its surrounding.

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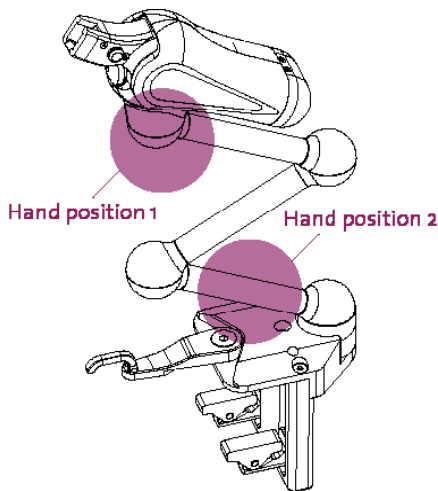
Figure 5: Indications of hand positions to detach Dowing²

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Detach Dowing²

STEP 1: Lock Dowing² in the parking position (see park or start using Dowing²)

STEP 2: Remove the user's arm

STEP 3: Remove Dowing² from the parking position. Warning: the arm fitting will move upwards due to the energy stored in the spring.

STEP 4: Remove the human interface (See attach and detach the human interface)

STEP 5: Hold with one hand the body of Dowing²; with the other hand, lift up in a straight line the lower swivel arm from the clamp

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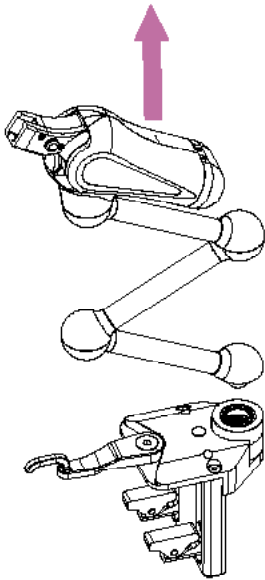
Figure 6: Take Dowing² apart from the table clamp or chair mount

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Attach Dowing²

- STEP 1: Position Dowing² above the clamp.
- STEP 2: Insert the lower swivel arm into the rotation head.
- STEP 3: Attach the human interface.
- STEP 4: Lock Dowing² in the parking position.

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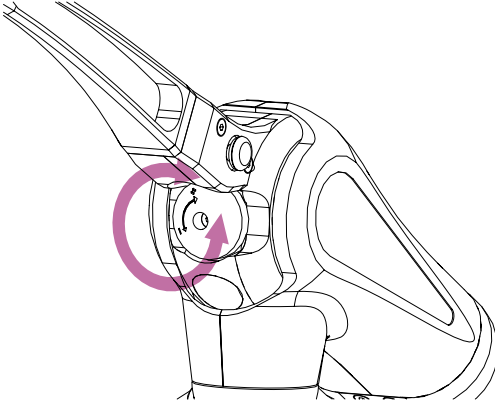
Figure 7: Adjust the balancing force
Figuur 7: Aanpassen balanceerkracht

The upwards balancing force can be adjusted by turning the adjustment knob. The force required to rotate the adjustment knob can change depending on the position of the human interface. This is normal behaviour. Because of this, the system can be adjusted using little force. Rotating the knob clockwise will give more balancing force, counter clockwise will give less force.

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E-mail: info@focalmeditech.nl

Internet: www.focalmeditech.nl

The complete user manual is available on our website and can be downloaded (<http://www.focalmeditech.com/downloads>). In case you want to receive a printed user manual, please contact us at +31 (0)13 533 31 03 and we will send you without any costs a hard copy within 7 days.

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the 1990s, the number of people with a disability has increased in all countries.

There are several reasons for this increase. First, the population is ageing. The number of people aged 65 and over has increased in all countries. Second, the number of people with a chronic disease has increased. Third, the number of people with a mental illness has increased. Fourth, the number of people with a physical disability has increased. Fifth, the number of people with a sensory disability has increased.

The increase in the number of people with a disability has led to a corresponding increase in the number of people who are dependent on others. This is because people with a disability often have difficulty performing daily activities and need help from others. This help can be provided by family members, friends, or professional care providers.

The increase in the number of people with a disability has also led to a corresponding increase in the number of people who are dependent on others. This is because people with a disability often have difficulty performing daily activities and need help from others. This help can be provided by family members, friends, or professional care providers.

The increase in the number of people with a disability has also led to a corresponding increase in the number of people who are dependent on others. This is because people with a disability often have difficulty performing daily activities and need help from others. This help can be provided by family members, friends, or professional care providers.

The increase in the number of people with a disability has also led to a corresponding increase in the number of people who are dependent on others. This is because people with a disability often have difficulty performing daily activities and need help from others. This help can be provided by family members, friends, or professional care providers.

The increase in the number of people with a disability has also led to a corresponding increase in the number of people who are dependent on others. This is because people with a disability often have difficulty performing daily activities and need help from others. This help can be provided by family members, friends, or professional care providers.

the 1990s, the number of people with a disability has increased in all countries.

There are several reasons for this increase. First, the population is ageing. The number of people aged 65 and over has increased in all countries. Second, the number of people with a chronic disease has increased. Third, the number of people with a mental illness has increased. Fourth, the number of people with a physical disability has increased. Fifth, the number of people with a sensory disability has increased.

The increase in the number of people with a disability has led to a corresponding increase in the number of people who are dependent on others. This is because people with a disability often have difficulty performing daily activities and need help from others. This help can be provided by family members, friends, or professional care providers.

The increase in the number of people with a disability has also led to a corresponding increase in the number of people who are dependent on others. This is because people with a disability often have difficulty performing daily activities and need help from others. This help can be provided by family members, friends, or professional care providers.

The increase in the number of people with a disability has also led to a corresponding increase in the number of people who are dependent on others. This is because people with a disability often have difficulty performing daily activities and need help from others. This help can be provided by family members, friends, or professional care providers.

The increase in the number of people with a disability has also led to a corresponding increase in the number of people who are dependent on others. This is because people with a disability often have difficulty performing daily activities and need help from others. This help can be provided by family members, friends, or professional care providers.

The increase in the number of people with a disability has also led to a corresponding increase in the number of people who are dependent on others. This is because people with a disability often have difficulty performing daily activities and need help from others. This help can be provided by family members, friends, or professional care providers.